

Measuring Technique

The measuring tool shows our three exact length sizes and is very pliable to help achieve an accurate measurement for your patient. When measuring, his/her lips should be at rest and slightly open. The 60 and 65mm lengths are the most popular. Using the measuring tool (most will start with the 60mm printed on the tool), go from commissure to commissure curving just beneath the vermilion border.....rather than straight across. Selecting the size of the implant, as well as placement is key to the success of the procedure. Please note: our 55mm length is the most uncommon length used. Without exceeding the length of the lip, you want the implant to be placed commissure to commissure.

After a few times using the tool it becomes very easy. It's not uncommon for the upper lip to be slightly longer than the bottom due to the curvature of the lip; however, this is not always the case.

Disclaimer: The intended use of the PermaLip™ Product is not approved for distribution within the United States.

